

AIR~SUPPORT CELLULAR AIR BED

Alternating air mattress system



Proven against pressure ulcers, the Hawksley alternating cellular mattress system is effective and comfortable as well as being rugged and reliable. Priced at a level that makes it economic to be put on every long stay bed, the Air~Support mattress will deliver huge savings by reducing pressure injury risks for your patients.

- Cellular alternating pressure mattress
- Tough, comfortable, easy-to-clean material
- Reliable and quiet air pump system
- Simple, easy to operate controls.
- Suitable to be hung on a standard NHS bed
- Capable of supporting patients up to 100kg
- Effective for reduction of pressure ulcers, venous insufficiency and swollen limb conditions
- Rental and lease options available

AIR~SUPPORT CELLULAR AIR BED

Alternating air mattress system

Technical specifications

Cellular mattress


Colour	Light blue
Height inflated	55mm (2")
Dimensions	200cm Long x 91cm Wide (deflated) c/w tuck-in flaps
Construction	Non-toxic stain-resistant PVC plastic
Warranty	12 months

Air pump

Cycle time	6 minutes
Pressure range	40mm Hg to 130mm Hg
Power cord	1.8 metres
Colour	Light grey/green base
Dimensions	11cm height x 22.5cm deep x 10cm wide
Weight	1.3kg
Construction	Impact-resistant plastic shell and base
Warranty	12 months


Ordering information

19000-00	Air~Support cellular air bed 220V c/w mattress
19001-00	Cellular mattress only

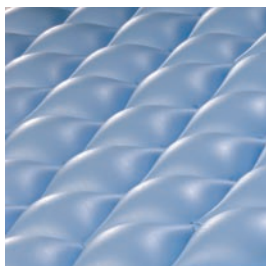


Alternating pressure cycles

Mattress cycles by inflating the A and B cells shown below in a total cycle time of 6 minutes broken into four phases.

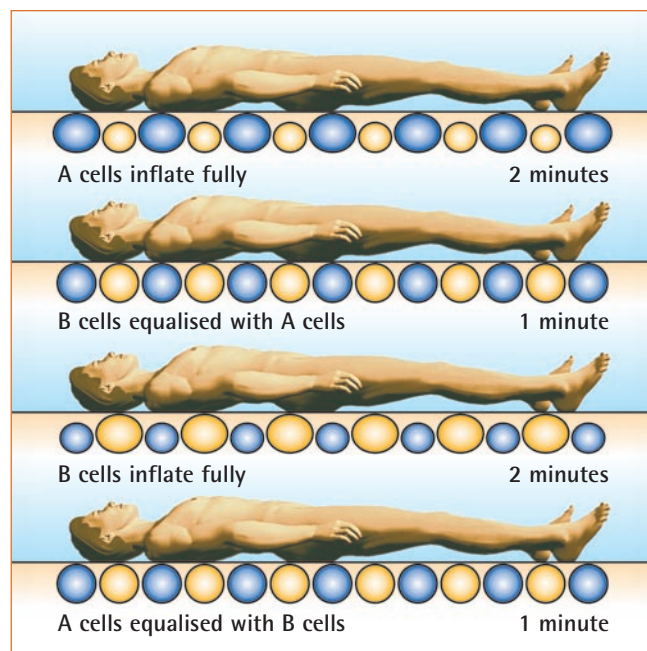

Typical weight distribution of adult male

Alternating pressure relief redistributes the pressure on the patient to allow better blood circulation.

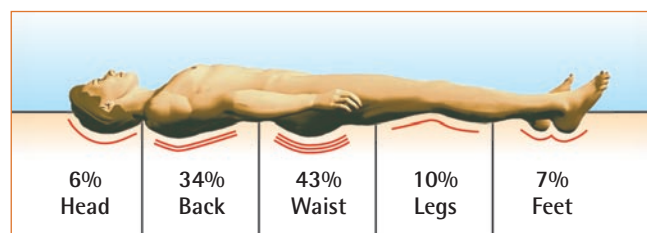

Hawksley • Medical and Laboratory Equipment

Marlborough Road, Lancing Business Park, Lancing, W. Sussex BN15 8TN, UK

T +44 (0)1903 752815 • F +44 (0)1903 766050 • www.hawksley.co.uk

